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-5-

Introduction

Thank you for choosing Excelled Curing Light. For the safety and for the sake of having the best result, please read the user's guide carefully before using the product. This user guide describes the normal properties of the Excelled Curing Light, and introduces the principle of the safety issues, installation, usage and maintenance.

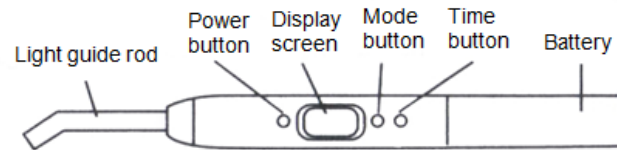
We are always trying to improve our products, this may cause some changes of the components, this is acceptable. When you find the product is different with the description in the user guide, please contact

the distributor or contact our service center to know the details.

Before the first use, please check the accessories; make sure they are all in good condition.

• Main unit	1 set
• Light guide	1 piece
• Battery	1 piece
• Charge	1 set
• Adapter	1 piece
• Light cover plate	1 piece
• User manual	1 piece

Structure

USAGE

1. Battery charge

connect the adapter to the AC power and connect then adaptor to charger. Place the hand piece on the charger. When charging, the charging indicator (yellow) is on, the power indicator (green) is off. When the battery is fully charged, the charging indicator (yellow) is off, the power indicator (green) is on.

When using a new battery for the first time, it is better to charge it for 6-7 hours, but should not be charged for over 7 hours.

When battery is full charged, please disconnect the AC power.

-6-

2. Operation

A) Power on/off: when the unit is off or standby, press power button to make the unit working, the light output shows a blue light, press the button again to stop working and turn to standby.

B) Mode select: press mode button to select three types of work mode:

- 01 gradually mode
- 02 flashing mode
- 03 all light mode

C) Time select: press time button to select time.

Four types of time setting: 10/20/30/40 seconds can be chosen.

D) When the unit is working, the buzzer will beep every 5 seconds.

E) Automatically power down: when the unit is standby and without pressing the button for 8 minutes, the unit will be turned off automatically.

F) Low battery alert: If starting the unit power and the voltage of the battery is lower than the set point the unit will give an alarm; the buzzer beeps 5 times and the indicator flashes. Please charge the battery in time.

SPECIFICATONS

Size	dia 22*175mm (unit)
Weight	165 g
Safety class	Ordinary type B, non-continuous service, IP 40, Class II
Input voltage	100-240 V, 50-60 HZ
Battery capacity	2200mAH
Wavelength	430-485nm
Intensity	5W>1100mw/cm2
Solidification effect	20s, > 3mm (3W), 6s 3mm (10W)

WARNING

- Only use original parts
- Protect your eyes when using device
- Prevent from burns when using extensively

NOTICE:

1. The working time between each handling is not less than 30 seconds.
2. Please replace the light guide when it has cracked or show scratches.
3. Disinfection of the light guide: immerse the rod in 75% alcohol several minutes.
4. When the battery is fully charged, please disconnect the AC power.
5. Avoid touching the electrodes of the battery with metal.
6. Please use a swab with anhydrous alcohol to clean the pins in the charger regularly, it could avoid weak contacts from charger to hand piece.

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